
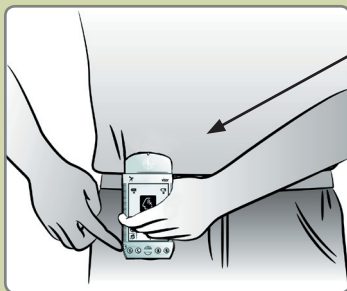


Requesting a Dose:

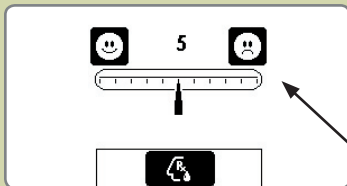

1 Turn your Personal Therapy (PTM) Manager ON


2 Select the Dose Activation Request Button

3 Make a patient diary entry, if your doctor has enabled this function

4 When the icon appears, position your Personal Therapy Manager over your pump as shown

5 Be patient – communication with the pump can take more than a minute

6 Confirm the status of your activation request

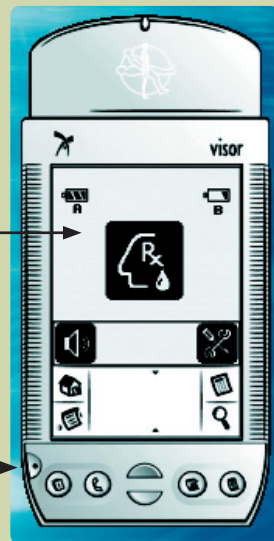

Possible Issues:

- You can't successfully activate a dose



- The Communication Light is orange

- You can't see the Communication Light or the Battery Warning appears



- The pump alarm is sounding



- You are unable to correct an issue or you are unable to turn the PTM on

Explanation:

- You may be in a lockout interval. Wait and try again or

- Your PTM did not communicate with your pump. Reposition and try again.

- Reposition the PTM until the Communication Light turns yellow

- Replace the PTM and/or Programming Module batteries

- Check your Pump Alarm Status (see your Patient Manual) and **call your doctor with the message screen number**

- Reset your PTM (see your Patient Manual)

Patient Activation Request Status Screens



Successful Dose Activation



Dose Activation Request Denied



Communication Failed, reinitiate activation request

Personal Therapy Manager

Patient Reference Card



UC200600967 EN NP6871
PNM1513
© 2005 Medtronic, Inc.
All Rights Reserved



	Speaker ON Press this button to turn the speaker OFF
	Speaker OFF Press this button to turn the speaker ON
	Move PTM The PTM cannot find your pump - move the PTM around the area of your pump
	Interrogate Pump Press when you want the PTM to communicate with your pump
	Return to Home Screen Takes you to the main dose request screen (see above)

Information Screens

Battery Empty
Replace batteries now

Battery Low
Replace batteries as soon as possible


Dose Request Success
Dose Activation has been confirmed

Dose Request Denied
You are attempting to activate a dose during a lockout interval.

Communication Failed
Wait and try again.

Searching for Pump
Position your PTM over your pump

Attention
Appears with a code - read this to your doctor

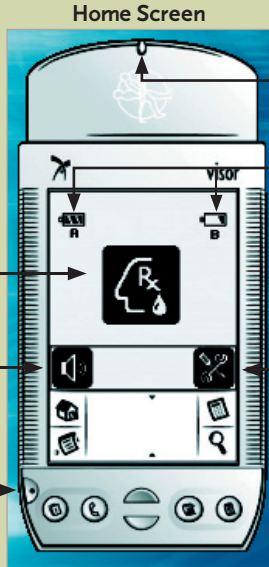

Dose Activation Request Button

- Requests activation of a dose supplemental to the continuous infusion of medication supplied by your implanted infusion system

Speaker ON/OFF Button

- Turns the PTM tones ON or OFF

Power

- Turns the PTM ON and OFF

Communication Light

- Blinks while the PTM is searching for your pump
- Maintains a steady light when the PTM is communicating with your pump

Battery Level Icons

- Tells you how much power is left in the (A) PTM and (B) Programming Module Battery

Toolkit Button

- Takes you to the Toolkit screen
- Provides access to therapy information and PTM status and settings