


# TARGETmyStim™


## Quick Guide Supplement


### Medtronic myStim® Patient Programmer Model 37743



#### Using TARGETmyStim™

1. Hold the patient programmer over the neurostimulator and press the NEUROSTIMULATOR ON  key.

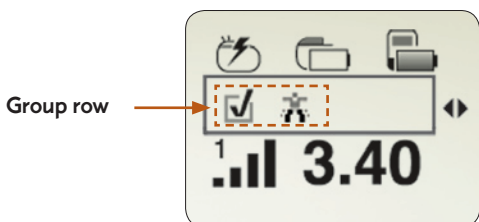


2. Verify the appropriate group is active  on the Group row.




3. Use the up  and down  NAVIGATOR keys to select the Parameter row.



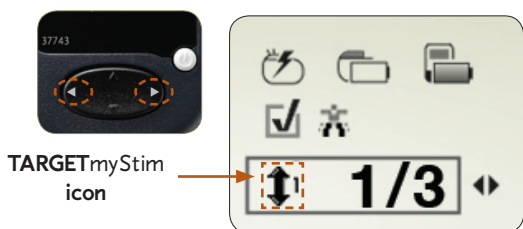


4. Use the left  and right  NAVIGATOR keys to select the appropriate TARGETmyStim icon ().

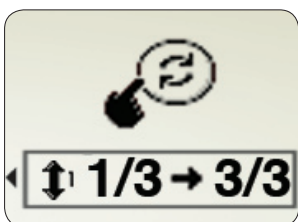


5. Press the Increase  or Decrease  key to display the initial **TARGET**myStim Screen.



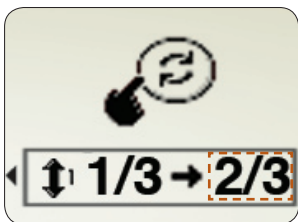

6. To change active electrode positions as needed, press the Increase  or Decrease  key to move stimulation to the desired position.



7. Once the desired position is displayed, hold the programmer over your neurostimulator and press the Sync  key to change to the selected electrode position and to slowly increase the amplitude.


*Note: A series of beeps will be heard while the amplitude is increasing.*


Pressing the Increase  or Decrease  key will stop the amplitude increase before it reaches the programmed value.



Sync Key



8. Press the left  NAVIGATOR key to return to the Therapy Screen.

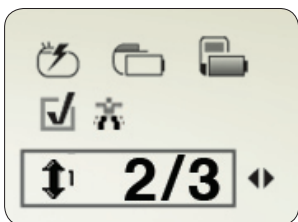

See your patient programmer user manual for complete instructions and information on warnings, precautions, risks, and benefits. Rx only.